

Instruction for use

Please read this information carefully because it will help you to use HYCOSAN® Night properly.

HYCOSAN®

Night

For improvement of the tear film and protection of the eye's surface in case of dry eye disease

Eye ointment without preservative

PRODUCT DESCRIPTION

HYCOSAN® Night is a sterile eye ointment with Vitamin A for ocular use.

COMPOSITION

1 g HYCOSAN® Night contains 250 I.U. retinol palmitate (Vitamin A) and liquid paraffin, light liquid paraffin, wool fat, white soft paraffin.

1 dosage = 1 strip of eye ointment (approx. 1 cm long) corresponds to approx. 5 I.U. Vitamin A.

One tube contains 5 g of eye ointment.

INTENDED PURPOSE

HYCOSAN® Night improves the tear film and protects the eye's surface in case of dry eye disease.

MEDICAL INDICATION

Dry eye disease

SUITABLE FOR

HYCOSAN® Night is suitable for patients of all ages. Based on existing knowledge, children, pregnant and breast-feeding women can also use HYCOSAN® Night.

BENEFITS, PERFORMANCE AND EFFECTIVENESS

The clinical benefit is the relief from symptoms of dry eye disease.

HYCOSAN® Night is a particularly soft and smooth eye ointment that spreads very well on the eye's surface. Vitamin A is a natural component of the tear film supporting the eye ointment's good tolerance and helps the ointment to mix well with the available tears. Thus, the sensation of burning, dry or tired eyes is relieved. The eye lids slide easily over the surface of cornea and conjunctiva.

Air conditioning, wind, cold, direct sunlight, pollution or cigarette smoke often cause dry or itchy eyes with a feeling of discomfort or foreign bodies.

Working in front of a screen for long periods, watching TV or driving at night can also cause these symptoms. If you are wearing contact lenses, there is an even higher risk for experiencing these symptoms.

HYCOSAN® Night relieves these symptoms and provides convenient long-lasting protection.

The consistency of the ointment causes a slight impairment of vision. Therefore, HYCOSAN® Night should be applied before going to bed. Possible discomfort when opening the eyes for the first time in the morning can be prevented by using HYCOSAN® Night.

GENERAL INSTRUCTIONS FOR USE

When using eye ointments, contact of the tube-tip directly with the eye or the skin should be avoided.

Only use HYCOSAN® NIGHT if the seal of the carton was intact before the first use (☞).


Wash your hands and stay in front of a mirror.

Unscrew the cap covering the nozzle.

Apply the eye ointment by gently squeezing the tube without folding or rolling.



Tilt your head backwards and gently pull on the lower eyelid using a finger of one hand.

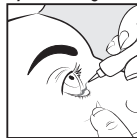

With the other hand hold the tube in an upright position above the eye. Apply one strip of eye ointment into the gap formed between the eye and eyelid by gently squeezing on the tube.



Close your eye gently then blink normally.

Replace the cap firmly back onto the tube.

HYCOSAN® Night can be used in combination with lubricating eye drops. HYCOSAN® Night should be used about 30 minutes after the application of the eye drops. In case you need to use a further eye ointment please keep an interval of at least one hour between both applications.

HYCOSAN® Night can be used for 6 months after first opening and should not be used beyond expiration data (📅). Please contact the manufacturer, your pharmacist or ophthalmologist in case of any questions about correct use of this medical device.

DOSAGE

The frequency of use depends on your personal needs. HYCOSAN® Night is used once a day before going to bed. In more severe dry eye cases, HYCOSAN® NIGHT can be applied three times a day into each eye. If you feel the need to use HYCOSAN® NIGHT more often, speak with your doctor.
Suitable for long-term use.

If you follow the dosage instructions, overdosing with HYCOSAN® NIGHT is unlikely as the target tissue is the ocular surface especially the conjunctival sac which has a limited volume.

PRECAUTIONS AND WARNINGS

If you do not feel better or if you feel worse after several days of use talk to your doctor or pharmacist.
If you use HYCOSAN® Night during the day, your vision will be impaired for some time because of the fatty consistency of the ointment. Until this clears up, you should not drive a car, use machines or work without stable footing.
HYCOSAN® NIGHT is intended to be used by one person and should not be shared with others (👤).

SIDE EFFECTS

The ingredient wool fat may cause localized skin reactions (e.g. contact dermatitis). Please check before use if you have a known hypersensitivity to wool fat.
In very rare cases hypersensitivity reactions like red, burning, itching eyes, excessive tearing or other temporary local irritations have been reported, these symptoms improved after the treatment with HYCOSAN® Night was stopped.

If a serious incident has occurred during the use of this medical device or as a result of its use, please report it to the manufacturer (🏭) and to your national authority.

CONTRAINDICATIONS

- HYCOSAN® Night should not be
- applied if you are hypersensitive to one of its ingredients
 - used while wearing contact lenses.

- used directly after ophthalmo-surgical interventions or in acute injuries of the eye as no clinical studies confirm this indication.

STORAGE

Do not store above 25 °C (🌡️ 25 °C).

Keep HYCOSAN® Night out of the reach and sight of children.

Any medical device you no longer use should be thrown away via household waste. This measure will help to protect the environment.

Date of information:

September 2025

🏢 URSAPHARM Arzneimittel GmbH
Industriestraße 35
66129 Saarbrücken
Germany
www.ursapharm.de

👤 and UK Responsible Person:
Scope Ophthalmics Ltd.
1st Floor, Unit 4 Amberley Court
Whitworth Road, County Oak Way
Crawley, West Sussex, RH11 7XL
Freephone UK: 0808 175 3200,
Freephone IRE: 1800 816005
Fax: +44 203 515 9461
Email: info@scopeeyecare.com
Website: www.scopeeyecare.com

CE 0197

MD

STERILE A

Legend of pictograms:

Symbol	Explanation
🏢	Distributor
MD	Medical device
CE	CE mark
UDI	Unique Device Identifier printed on the folding carton.
STERILE A	Sterilized using aseptic processing techniques
LOT	Lot number
📅	Date of manufacture [YYYY/MM]
📖 i	Consult instructions for use